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SKIN PEEL

DERMOPEELING E NON SOLO

SKIN

PEEL

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PEELING AND BIO-STIMULATION

WITH THE SAME PRODUCT:

TRADITIONAL PEELING AND BIO-STIMULATION.

TWO METHODS FOR SKIN REPARATION

WIDELY ADOPTED IN AESTHETIC MEDICINE,

USED IN DIFFERENT TIMING AND IN DIFFERENT CONDITIONS,

BUT WITH **A UNIQUE COMMON USE:**

**ELIMINATING OR REDUCING THE IRREVERSIBLE
OUTCOMES OF PREVIOUS PATHOLOGICAL
CONDITIONS, (ACNE, SEBORRHEIC SKIN, KERATOSIS,
DYSCHROMIA) AND PHOTO-AGEING.**

**RESTORING VOLUME, TURGOR
AND BRIGHTNESS OF THE SKIN
THANKS TO RENEWED TISSUE HYDRATION.**

SKIN PEEL

M

Medical Device class I, with Mandelic acid (55%)
- α -hydroxy acid (AHA) - exfoliating chemical
extremely handy and not photosensitizing.

It induces, through **forced exfoliating**, the removal of the **superficial skin stratum** with contemporary release of inflammatory cytokines. The cytochemical cascade activates the cell turnover, renewing skin tissue.

EXFOLIATING AND LEVIGATING ACTION FOR FREQUENT USAGE, SUITABLE FOR EVERY SEASON DUE TO THE ABSENCE OF PHOTSENSITIZATION.

SKIN PEEL

G

Medical Device class I, with glycolic acid (70%) - α -hydroxy acid (AHA)- exfoliating chemical extremely handy and not photosensitizing and SKIN-B.

Glycolic and **mandelic acid** are among the most AHA used in the world of aesthetics. **Glycolic acid**, consisting of very small molecules, easily penetrates the skin, facilitating the elimination of the most superficial layers of the epidermis and preparing it for bio-stimulating and regenerative action, exercised by the simultaneous presence of SKIN B into the formulation.

SKIN B is a unique compound, consisting of fragments of hyaluronic acid and dermostructural amino acids, able to stimulate the fibroblasts to produce new matrix.

**EXFOLIATING
LEVIGATING, BIO-STIMULATING
AND REGENERATIVE ACTION.**

SKIN PEEL

JESSNER

Medical Device class I, with resorcinol, lactic acid (15%) salicylic acid (15%) and SKIN-B.

A complete formulation exploited by every single component:

Resorcinol: favors the separation of the stratum corneum and lightens the skin.

Lactic Acid: normally present in the hydrolipidic mantle, performs exfoliating action, it regulates the keratinization and the physiological PH.

Salicylic acid: reduces the skin blemishes, acting on the imperfections, giving brightness also thanks to the sebum production regulation.

SKIN B is a unique compound, consisting of fragments of hyaluronic acid (20-38 monomers) and dermostructural amino acids, able to stimulate the fibroblasts to produce new matrix.

**EXFOLIATING,
LEVIGATING, LIGHTENING,
BIO-REGENERATIVE,
RESTRUCTURING ACTION.**

SKIN PEEL COMPLEX

Medical device class I based with Mandelic Acid (15%),
Salicylic Acid (8%) Pyruvic Acid (8%) and Skin-B .

Mandelic Acid (55%) - α hydroxy acid (AHA):
chemical exfoliant extremely handy and not
photosensitizing.

Salicylic acid: β hydroxy acid, reduces skin
blemishes, correcting the skin imperfections
giving brightness thanks to the Sebum
production regulation.

Pyruvic acid: if used alone or in association
with other substances such as Glycolic acid it
improves exfoliating action such as glycolic
acid it strengthens antibacterial action.

SKIN-B is a unique product constituted
of fragments of hyaluronic acid (20-38
monomers) and dermo structural amino
acids which are capable of stimulating the
fibroblasts to produce new skin matrix.

**THE INTEGRATED ACTION
OF THE THREE ACIDS WITH
COMPLEMENTARY SYNERGY,
WHICH IN TERMS OF
CONSISTENCE IN THE FINAL
RESULT IS BRIGHTNESS
OF THE SKIN.**

SKIN

ONE ONLY TREATMENT WITH TWO
COMPLEMENTARY ACTION:
PEELING AND BIO-STIMULATION

SKIN PEEL M

Components:
Mandelic acid 55%
propylene glycol.

Indications:
seborrhea, acne, rosacea,
post-inflammatory
hyperpigmentation,
freckles, lentigo solaris,
melasma, moderate
photo-ageing.

SKIN PEEL G

Components:
Glycolic acid 70%, Natrosol,
Sodium Hyaluronate,
L-Isoleucine, L-Leucine,
L-Lysine hydrochloride,
L-Proline, L-Valine,
Glycine, L-Serine, L-Alanine,
imidurea EDTA tetrasodium.

Indications:
moderate photo-ageing,
active acne, scars,
rosacea papulo-pustolosa,
skin hyperchromias,
seborrheic keratosis.

METHODS OF USE OF SKIN PEEL M, SKIN PEEL G, SKIN PEEL JESSNER AND SKIN PEEL COMPLEX

It is important to deterge the face before use. Apply the solution with a fan brush, dividing the face in zones beginning from the forehead, then left side, right side and chin. Leave to act for a few minutes, keeping it under supervision in order to determine to exact action time that can differ from different skin types. Once the skin has reached a redness coloring. Wash away with abundant water and bicarbonate or with specific solvent.

PEEL

**FOUR FORMULATIONS
SPECIFIC FOR INDIVIDUAL
NEEDS**

SKIN PEEL JESSNER

Components:
denatured ethyl alcohol,
Resorcinol, Lactic acid 15%, Salicylic
acid 15%, water demineralized,
Sodium hyaluronate, L-Isoleucine,
L-leucine, L-Lysine hydrochloride,
L-Proline, L-Valine, Glycine, L-Serine,
L-Alanine, Cysteine,
Sodium bicarbonate.


Indications:
Moderate photo-ageing,
comedonic acne,
papular acne, pustular acne,
seborrhea skin, melasma,
hyperpigmentation and
post-inflammatory
or photo types V and VI.

SKIN PEEL COMPLEX

Components:
Danaturated alcohol,
propylene glycol, Mandelic acid
15%, Demineralized water,
Salicylic acid 8%, Pyruvic acid
8%, hydroxypropylcellulose,
Sodium hyaluronate,
L-Isoleucine, L-leucine, L-lisine,
L-valine, Glycine, L-serine,
L-Alanine, L-Cisteine,
Sodium bicarbonate.

Indications:
Moderate photo-ageing,
thin wrinkles, or of medium
depth, Seborreheic skin,
comendic acne, pustular
acne, popular acne,
rosacea, lentigo,
melasma and
hyperpigmentation
and post-inflammatory.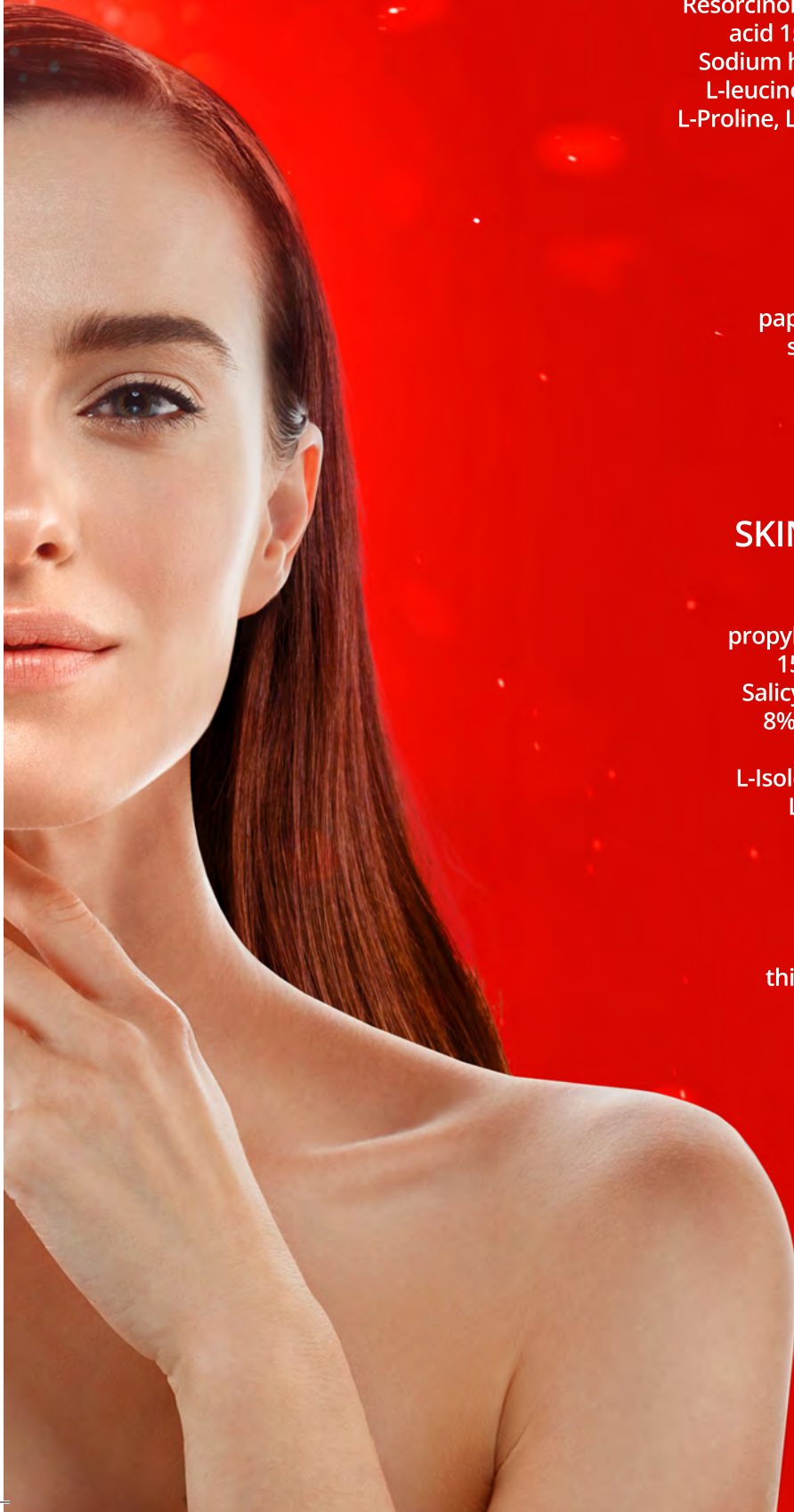

italfarmacia
LABORATORI

ALWAYS AHEAD FOR YOUR WELLBEING!

Leading Italian company in the research, creation and marketing of medical devices, supplements and cosmetic products. The Company operates in Italy since 1995 with its production site and since 2005, it has successfully inserted in the field of aesthetic medicine with the line of products that is the SKIN range, based on the principles of biostimulation, bioregeneration and tissue regeneration.

Italfarmacia has always placed its primary purpose innovation together with strict safeguarding of the quality of life, that life that is always getting longer and that it is essential to make as much as possible active, pleasant and worth living in all its aspects.

Beauty can overcome any age limit, with products and treatments that are natural, effective, long lasting and safe.

Via di Tor Sapienza, 7
00155 - Roma, Italy
www.italfarmacia.com

Phone +39 06 45441800
Fax +39 06 45441801
info@italfarmacia.com

Follow us on:

Rev. 02 of 04/03/2021